# drive

DeVilbiss®  
HEALTHCARE

# Samsoft 175

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## CONTENT

- I - Specifications** .....15
  - 1. *Samssoft 175*.....15
  - 2. Lifting assembly .....16
- II - Intended use** .....16
- III - Principles of assembly**.....16
  - 1. General terminology .....16
  - 2. Unpacking .....17
  - 3. Assembling.....17
  - 4. Folding.....18
  - 5. Separating the base from the rest of the patient lift .....18
- IV - Using *Samssoft 175*** ..... 19
  - 1. Commissioning.....19
  - 2. Charging .....19
  - 3. Possible problems .....19
  - 4. Battery pack.....19
- V - Transferring a patient**.....19
  - 1. Base spacing .....20
  - 2. Cross-bar .....20
  - 3. Transferring the patient from a bed to an armchair. ....20
  - 4. Transferring the patient from an armchair to a bed. ....20
  - 5. Transferring the patient lying on the floor.....21
  - 6. Moving.....21
- VI - Drive devilbiss healthcare france sling range**....21
  - 1. Controls.....21
  - 2. Cleaning.....21
- VII - *Samssoft 175* maintenance** .....22
  - 1. Cleaning.....22
  - 2. Disinfecting the patient lift .....22
  - 3. Daily maintenance.....22
  - 4. Routine maintenance.....22
  - 5. Preventive maintenance.....22
- VIII - Liability and warranty conditions** ..... 23
- IX - Before contacting Drive Devilbiss Healthcare France customer service** .....24
- X - Service book** .....25

- Only use the slings provided by Drive DeVilbiss Healthcare France.
- Maximum patient weight: 175 Kg.
- Carefully read the user guide before using this device.
- This device should be used by skilled and experienced staff.
- Never use a defective patient lift.
- Annual maintenance is compulsory.

## WARNINGS

The renter must inform and show the user how to handle the patient lift stating the precautions and the risks

related to inappropriate use. Pinching areas should be explained in details.

The renter should ensure the user perfectly understood the safety instructions.

When mounting and using the device, it is recommended to have the hands away from the pinching areas.

When the patient lift is exposed to heat source (sun rays through a window or radiator) for a long time, ensure there is no burn risk before using the patient lift.

The device should be used in a sufficiently clear place so that manoeuvres are not disturbed. Since the slings are not fireproof, it is forbidden to use the device near flames and to smoke.


Do not use or store the patient lift in a humid area or near water splashing (rain, shower, etc.).

In very particular use situations, the patient lift operation can be disturbed by radio and electromagnetic waves (proximity of radio-telephone antenna, etc.). In case of dysfunction, do not use the patient lift. Its operation can interfere with sensitive electronic devices. In this case, do not use the patient lift.

Because of the component weight and dimensions, there are hazards during dismantling and reassembling, so it is highly recommended being assisted and minding the pinching areas.


## SYMBOLS AND EXPLANATIONS

	Maximum user weight
	Caution: hazard!
	Carefully read the user manual
	Device compliant to directive 93/42/CEE
	This device is submitted to directive 2002/96/EC relative to waste electric and electronic equipment (WEEE) and it was marketed after 13.08.15.
	Patient lift manufacturer
	Serial number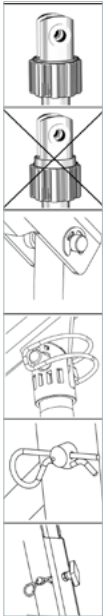

- Check the red emergency lowering ring is in the up position.
- Check the elastic ring is correctly engaged in the groove of the axle.
- Check the spring pin is correctly oriented on either side of the lug and correctly closed.
- Check the beta pin in the lower part of the mast is correctly positioned.
- Check the wing knob is correctly screwed in.

## IDENTIFICATION LABEL

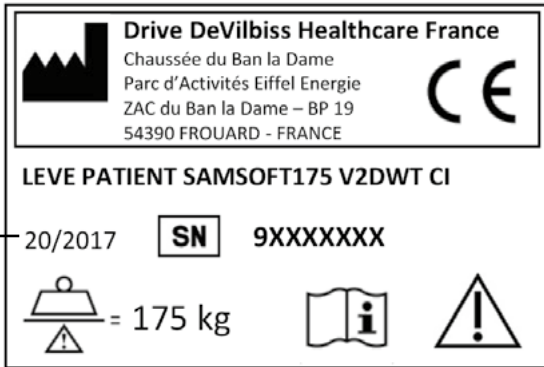Manufacturing date (week/year)

## I - SPECIFICATIONS

### 1. *Samsoft 175*


<b>Dimensions (folded)</b>	Lxlxh 130x63x40 cm
<b>Dimensions (unfolded)</b>	Lxlxh 133x63x134 cm
<b>Lifting capacity</b>	175 kg
<b>Inner feet spacing</b>	54 cm min to 83 cm max.
<b>Turning diameter</b>	144 cm
<b>Base height</b>	9 cm
<b>Lifting time</b>	38 s (empty)
<b>Patient lift weight</b>	38,4 kg
<b>Lifetime</b>	10000 cycles at 175 kg
<b>Front wheels</b>	∅ 7,5 x 6 cm without brake
<b>Rear wheels</b>	∅ 7,5 x 6 cm with brake

<b>Structure</b>	Steel
<b>Coating</b>	Epoxy RAL 7005 - 7004 - 8000N
<b>Electric cylinder (lifting)</b>	24 V continuous 8000N
<b>Mechanical cylinder rod disengagement (red ring)</b>	
<b>Battery power supply</b>	2 x 4.5 Ah, 12 V (included in the battery pack)
<b>External battery charger</b>	
<b>Battery charge indicator</b>	Par LED
<b>Buttons</b>	Emergency stop and emergency lowering
<b>Cycle number as for patient lift autonomy (lifting/lowering) with an average weight of 100 Kg, with new charged batteries.</b>	80 cycles
<b>EC marking date</b>	2014

Complies with ISO 10535 standard


**Important:** the patient lift should be used with slings (safety members) provided by Drive DeVilbiss Healthcare France exclusively.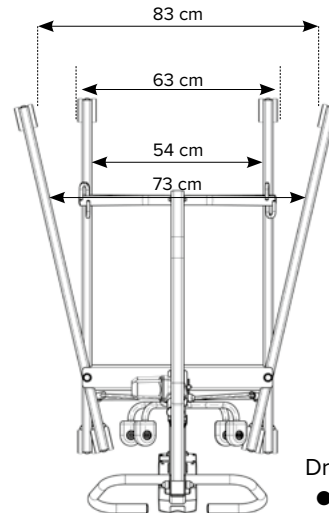

Driven operating force :

- with one finger : ..... < 5 N
- with one hand : ..... < 105 N
- with one foot : ..... < 300 N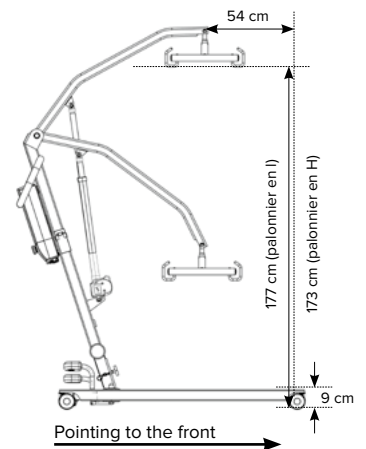

Dimensions +/- 1 cm

## 2. Lifting assembly

**Autonomous box :** (control box + battery pack)

Dimensions	L=368 mm l=120 mm h=107,6 mm
Weight	4,8 kg
Protection	Current limiter
Output voltage	24 V continuous
Output current	8,5 A maxi
Output number	1
Battery voltage	2 x 12-V batteries
Battery capacity	4,5 Ah
Autonomy	Around 80 cycles
Charging time	8 h
Emergency stop push-button/ Emergency lowering button	On the control box
Battery indicator	Charged/discharged
Protection index	IP 54

**Actuator of cylinder :**

Power supply	24 V continuous
Max center to center distance	895 mm
Max. stroke	300 mm
Max. thrust	8000 N
Lifting time when empty	38 s
Lifting time when a patient is in	54 s
No-load speed	26 mm/s
On-load speed	19 mm/s
Weight	3,5 kg
Protection index	IP X4
Mechanical decoupling / Safety nut	
Sound level	≤ 60 dB

**External battery charger:**

Power supply	230 VAC 50-60 Hz
Output	24 V continu
Protection index	IP 20

## CERTIFICATION

*Samssoft 175* complies with ISO 10535 standard (March 2007).

## II - INTENDED USE


● This transfer device is intended to transfer mobility-impaired patients; it can be used with a wheelchair, a shower seat, a bed, or a toilet seat.

**Not suitable for children under 12 year-old, for patients with mental disorders, and for patients whose morphology is incompatible with this kind of transfer device**

## III - PRINCIPLES OF ASSEMBLY

### 1) General terminology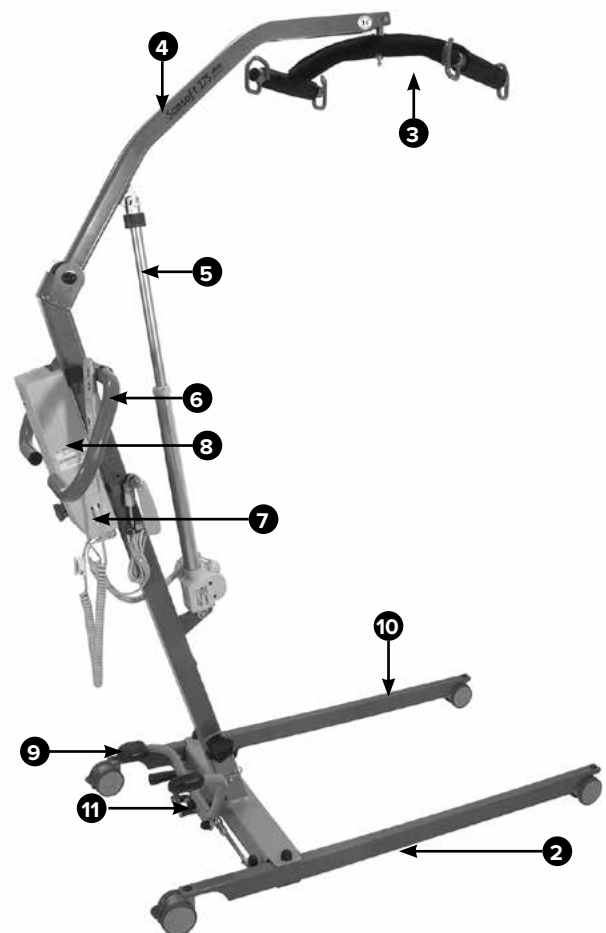

- |                      |   |
|----------------------|---|
| 1 - Mast             | 8 - Battery pack  |
| 2 - Base             | 9 - Spacing pedal of the side rails                                   |
| 3 - Cross-bar        | 10 - Side rail  |
| 4 - Boom             | 11 - Locking system of the side rails in the open position (optional) |
| 5 - Lifting cylinder |   |
| 6 - Remote control   |   |
| 7 - Control box      |   |

## 2) Unpacking

Check all the device elements have not been damaged during transport. Check the accessories are present: the lifting assembly (box with battery pack, cylinder, remote control), the battery charger, a user guide, and possibly a sling if it was ordered.


**NB: our device is mounted and tested in our manufacturing site, but its packaging requires it to be folded**

## 3) Assembling

### A - Provided parts:

- 1 - *Samsoft 175* structure with lifting assembly: box (including battery pack), cylinder and remote control, battery charger.
- 2 - *Samsoft 175* user guide.
- 3 - Possibly one sling if it was ordered.

*Samsoft 175* when assembled and folded:
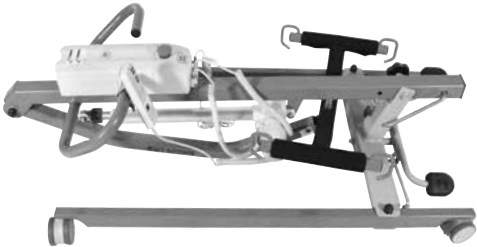


**⚠ CAUTION: before using the device, make sure all the shafts, screws, and pins handled during the installation are perfectly engaged in their respective supports, and they are correctly locked.**


*Samsoft 175* is functional provided the sling compatible with this device is used and provided the maximum permissible load (175 Kg) is respected.

### B - Structure installation

- 1 - Remove the structure from the box (possibly by cutting the box angles), lay the structure on a flat floor, and then lock the brakes and wheels.
- 2 - In the lower part of the mast, after removing the pin, remove the mast safety shaft.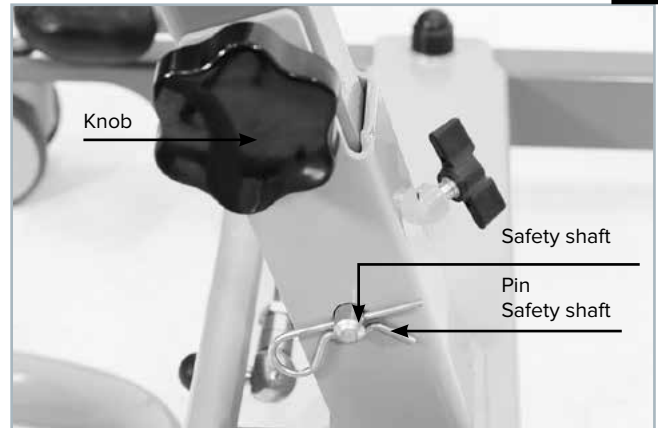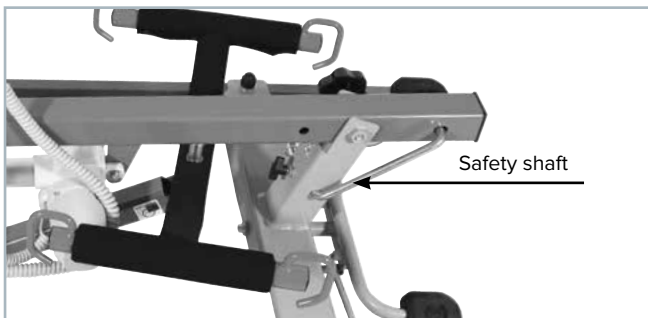

3 - Straighten up the mast, and then position the mast safety shaft into the second hole and reposition the pin. Tighten the backlash screw (black wing screw).

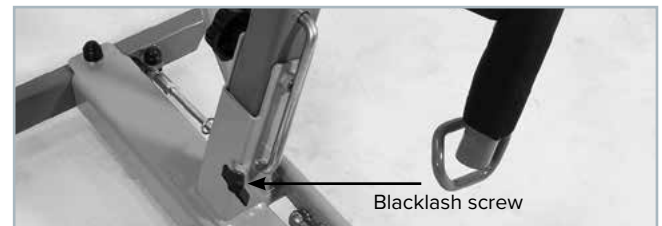
4 - With the remote control, get the rod out of the cylinder in order to unfold the boom. The ball joint linking the cross-bar to the mast automatically disengages

5 - Check the box is correctly charged:

- Green: the batteries are totally charged.
- Green and yellow: 3/4 charged batteries.
- Yellow: 1/2 charged batteries.
- Yellow and red: the batteries are almost discharged, recharge them
- Red: the batteries are discharged, they should be immediately charged otherwise their service life could be reduced.
- If the red indicator is lit up (under the small key), the device requires special maintenance, please contact drive DeVilbiss Healthcare France Customer Service.
- When you hear several repeated beeps when the patient lift is rising, it is a warning indicating the batteries are low.

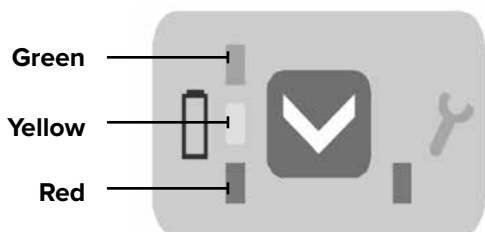
6 - If there is free play in the pedal when using the patient lift, you can adjust it with the rods.

## Supply box

(including the battery pack)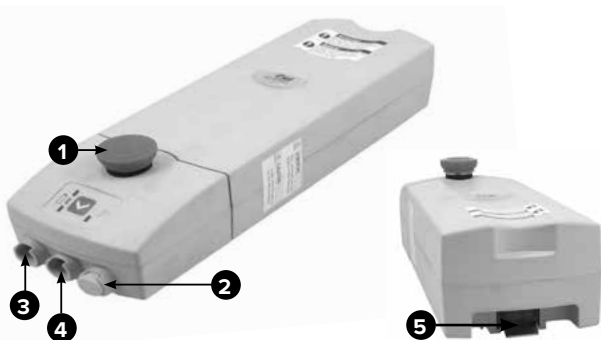

- 1 - Emergency stop push-button
- 2 - Emergency lowering button
- 3 - Cylinder connection plug
- 4 - Cylinder connection remote control
- 5 - Battery pack unlocking knob

## 4) Folding

The device should be folded to be transported.

- Press the remote control to refold the boom until the cross-bar ball joint is facing the mast clip.
- Snap the ball joint into the mast.
- Remove the pin and the mast safety shaft.
- Lower the assembly mast/boom/cross-bar onto the side rails.


CAUTION: pinching risk

- Reposition the mast safety shaft in its 1st position, and put its pin back.
- The device can be vertically transported or stored.

## 5) Separating the base from the rest of the patient lift

To easily carry *Samsoft 175*, you can disassemble it into 2 parts by totally unscrewing the knob and removing the safety shaft.

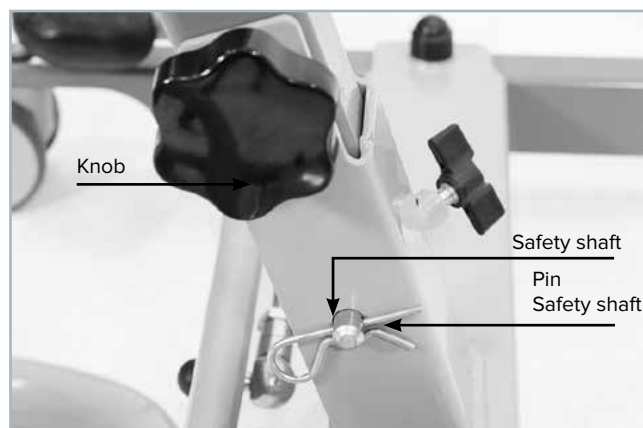**⚠ CAUTION :** Partially unscrew the backlash screw before separating the mast from the wheel base. After reassembling, check all the fastening parts and the safety pin are correctly installed. Firmly tight the knob.

Wheel base weight: 15,8 kg

Weight of the mast, the boom, and the cross-bar: 22,6 kg

## IV - USING *Samsoft 175*

**⚠ WARNING:** *Samsoft 175* should be used indoors exclusively on a flat and adherent floor; it should be handled by a competent caregiver trained for using this kind of devices.

### 1) Commissioning:

When all the connections are made :

- Use the patient lift with the remote control.
- Check the battery charge (see colour code in Chapt. III. 3 - B-5).
- The patient lift is ready to be used.

### 2) Charging:

**⚠ The power unit should be charged every night. It should be charged in a well-ventilated room. Check the power supply cord condition before plugging it.**

● There are 2 methods to charge this patient lift battery: the battery can be charged on its base or directly from the mains outside from the patient lift.

To charge the battery, you can plug it directly to a mains outlet, or you can remove it from its support and plug it to a mains outlet in order not to move the patient lift. During charging, the green indicator is blinking.

● When the green indicator is fixed, the battery charge is finished (see colour code in Chapt. III. 3 - B-5)

● The patient lift is ready to be used again.

### 3) Possible problems

- If the remote control cannot lower the patient, press the emergency lowering button
- If the indicator is steady red when lowering or lifting the patient, charge the device

● Check the connections if the remote control or the cylinder does not work

In case of problem when lifting or lowering a patient, press the emergency stop button, and then reload it by pulling it.

In case of power supply problem, you can manually lower or lift the boom by pressing and turning the red ring located at the end of the cylinder rod.

Do not forget to reposition it.

**⚠ CAUTION:** The cylinder should only be used discontinuously: 2 minutes operating, and then 18 minutes resting. Do not exceed 2 minutes operating in continuous.

Any duty cycle exceeding the above cycle can damage the lifting assembly (cylinder/control box). This damage is excluded from the warranty.

### 4) Battery pack

The 2 batteries included in the box of the battery pack do not require any special maintenance. Though, they should be regularly charged to maintain their service life (at least every 3 months).

With a second battery pack (available on request), the device can work 24/24 hours.

## V - TRANSFERRING A PATIENT

Before positioning and transferring the patient, check that:

- The patient's weight is compatible with the maximum permissible load of the patient lift (175 kg maximum).
- The batteries are charged and the patient lift is fully operational.
- The floor is flat and stable.
- The patient lift brakes are functional
- Check the sling is compatible with *Samsoft 175* (Drive Devilbiss Healthcare France sling).
- Check the sling fasteners and the patient support elements are in perfect condition.
- In the maintenance book included in this user guide, check regular maintenance is performed once a year at least.
- If the patient is agitated, an accompanying person should secure the patient's movement.


Once these checks are performed, you can transfer the patient.

**⚠ CAUTION:** You should move the patient lift base apart to the maximum before lifting or moving the patient. Do not engage the wheel brakes.  
**Do not keep on lifting or lowering the patient if the 4 wheels are not on the floor.**  
 The lifting task should be very progressive checking the patient is well positioned into the sling and there is no tipping risk due to excessive overhanging.

## 1) Base spacing


Press on the spacing pedal to open or close the *Samsoft 175* base. Its new design significantly reduces the effort required to commission the device.

● **Locking system of the side rails in the open position:** if needed, you can lock the side rails in the open position.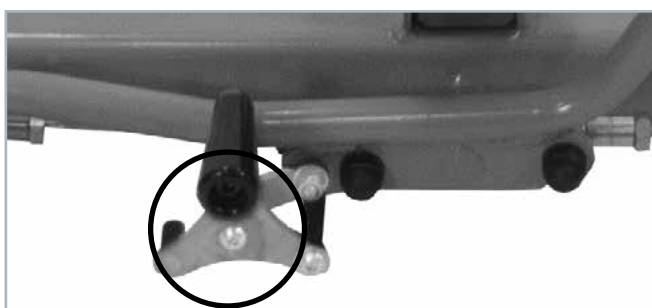

### LOCKING:

When the side rails are apart (opening pedal on the right side rail), lock the pedal pushing the locking handle downwards.

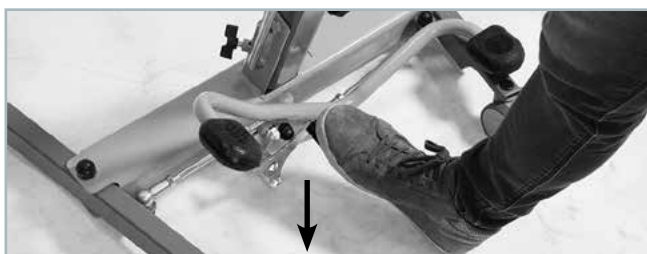### UNLOCKING:

Pull the locking handle upwards to unlock the pedal, then close the side rails pushing on the left pedal.

## 2) Cross-bar

This patient lift is equipped with a cross-bar with backstop clamps.

Before transferring the patient, check each buckle of the sling is within the clamps.

## 3) Transferring the patient from a bed to an armchair

- Position the patient on his/her side
- Place the sling under the patient.
- Check the back part of the sling is correctly positioned (from shoulders to coccyx).
- Lay the patient on the back.
- Pass each leg support under each thigh
- Position the patient lift at the patient's level
- Hook each sling buckle inside the crossbar hooks
- The patient can now be transferred from the bed to an armchair.

## 4) Transferring the patient from an armchair to a bed

- Lean the patient forward holding him/her.
- Slip the sling behind the patient's back up to the coccyx
- Pass each leg support under each thigh.
- Position the patient lift at the patient's level.
- Hook each sling buckle inside the crossbar hooks
- The patient can now be transferred from an armchair to a bed

Particular points :

- When positioning the sling, check there is no fold at the leg level in order not to hurt the patient.
- Do not engage the device brakes when lifting so that the device can correctly position with regard to the patient.

**CAUTION:**

In case of power supply problem, you can manually lower or lift the boom by pressing and turning the red ring located at the end of the cylinder rod.

## 5) Transferring the patient lying on the floor

The patient's weight should not exceed 175 Kg and the patient lift should be positioned so that the patient's head and trunk be located inside the side rails. Lift the patient very progressively ensuring he/she is well positioned on the slings.

## 6) Moving

Although this operation is strongly not recommended, it is possible to use the patient lift to move the hung patient on a very short distance (ex.: from the bed to the shower).

Use the handles to move and guide the patient lift. Another person should secure the patient.

- Do not push nor pull the slings to move the patient lift.
- Do not move a patient if agitated.

## VI - *Samsoft 175* SLING RANGE

Transfer slings for 4-point handle bar. Max. load 250 Kg. See the documentation specific to drive DeVilbiss Healthcare France slings.

Do not exceed the maximum load of the patient lift.

Bath net .....	M	size
Bath net .....	L	size
Comfort polyester .....	M	size
Comfort polyester .....	L	size
Bath netted hammock .....	One size	
Toilet netted hammock .....	One size	

### 1) Controls

(before any use, and to be performed by a competent staff)

Before any use, check the condition of the slings: no undone sewing, the upholstery and the slings should not be fringed, no tear and no wear sign (transparent upholstery weft).

### 2) Cleaning

(each time it is necessary, and to be performed by a competent staff)

- The polyester slings can be washed at 40° at the maximum. For other slings, see the sling instructions.

## VII - *Samssoft 175* MAINTENANCE

Disconnect the power supply cord and remove the battery pack before any maintenance operation.

### 1) Cleaning

(every month or when necessary, and to be performed by a competent staff)

- Clean the patient lift with a soft cloth dampened with soapy solution, then rinse, and carefully dry.
- Do not use aggressive detergent; do not soak.

### 2) Disinfecting the patient lift

(when necessary, and to be performed by a competent staff)

Disinfecting is performed with detergents and disinfecting solutions sprayed on the external chair areas.

It is a foaming solution that does not require rinsing, but only drying. Our internal services use ANIOS solutions.

### 3) Daily maintenance

Charge the battery pack every night.

### 4) Routine maintenance

(every month or when necessary, and to be performed by a competent staff)

The routine maintenance consists in checking the system functionality:

With the patient lift positioned on a flat floor, check its 4 wheels are touching the floor.

When the brakes are engaged, the patient lift should not be moved (flat and adhered floor).

Operate the patient lift with nobody in, check the arm and the side rails move steadily with no noise.

Control the handles located on the mast and check there is no play between the arm and the mast, and between the mast and the side rails.

Check the plastic protective housings and casings are not broken or cracked, and all parts are present.

Check the condition of the connecting cables: power supply cord, remote control and cylinder; if a cable shows a slight burn, wear, or impact sign, or if the wires are visible, immediately stop using the patient lift, and have the cables replaced by the engineering department.

### 5) Maintenance préventive

(every year or when necessary, and to be performed by a competent staff)

- It is necessary to control the device every year.
- The battery pack should be changed about every 18 months depending on how it is used.
- The crossbar suspension axle should be imperatively checked yearly on each maintenance task. Yearly maintenance should be performed by skilled technical staff experienced with electromechanical device maintenance. The maintenance tasks do not require

particular tools; adequate tools should be used according to the best industry practices. Mechanical fastening, plays, and adjustments should be realised according to the best industry practices

### 5.1) Mechanical control

- Check the welding. Visual inspection.
- Check the alignments. Visual inspection.
- Control and lubrication after dismantling the mast and side rail pivots.
- Control and lubrication of the cylinder fastenings.
- Check the tightening of all the parts.
- Dismantling, cleaning, and lubrication of the front and rear casters.
- Dismantling, cleaning, and lubrication of the mast hinge. Check the fastening, caster play, and brake performance. The mechanical parts that have cracks, wear signs or geometrical defects should be replaced with original components delivered by Drive DeVilbiss Healthcare France.

### 5.2) Electrical control

- Check the remote control.
- Check the charger.
- Check the remote control box and the cylinder.
- Check the lifting/lowering speed.
- Check the emergency stop and lowering.
- Condition of the electric cables (change the cables in case of doubt).

Replace the electric cables and/or the components to which they are connected if there are signs of wear, sheath cut, outlet damage or plastic and sheath darkening around the coupling area. Replace the electric cables and/or the components to which they are connected if there are signs of wear, sheath cut, outlet damage or plastic and sheath darkening around the coupling area. Never disassemble the protective housings of the cylinder, battery pack, remote control, control box, and battery charger.

### 5.3) Charging test

- Charge the battery pack
- Check the battery pack power and autonomy
- Test the device when fully loaded (175 Kg) and control the power cut threshold (230 Kg +/- 10 Kg).

### 5.4) Slings

The slings should be checked at least once a month, or more regularly in case of intensive use, ensuring the seams are not undone, the upholstery and sling webbing is not visible, and there is no torn part.

Never use a defective sling.

The worn or malfunctioning parts should be replaced with original parts manufactured by Drive DeVilbiss Healthcare France.

Have some tries with a load before commissioning the patient lift again.

## 5.5) Maintenance book

Every maintenance operation or intervention should be recorded in the service book attached to these instructions

## 5.6) Shipping, storage, and disposal

- The patient lift should be returned or transported in its original packaging.
- Before storing the patient lift, clean it totally and check its performance. Store the patient lift in a dry place away from dust and humidity. The batteries should be totally charged every 3 months
- When commissioning, after storing, charge the battery pack, and check its performance before using it again.
  
- Storage temperature : .....from 0 to + 50 °C  
..... Humidity between 10 and 80%
  
- Service temperature : .....from + 10 to + 40 °C  
..... Humidity between 30 and 80%
  
- The disposal of defective parts, packaging and battery pack should be performed by approved waste treatment centres.
- The involved system should not be disposed of with household waste, but should be sent to an appropriate waste collection point or to a distributor.
- By respecting these instructions, you respect the environment and participate in protecting the natural resources and the human health.

## VIII - LIABILITY AND WARRANTY CONDITIONS

● ***Samssoft 175*** is warranted for 5 years as far as the structure, the cylinder and the control box are concerned, and in case of normal use according to the instructions given in this guide. Negligence, handling errors and accidents are excluded from the warranty. The remote control is guaranteed for 3 months, and the battery pack is guaranteed for 6 months.

● **Any modification or addition of accessories without drive DeVilbiss Healthcare France's written consent will de facto exclude the patient lift from the warranty and from drive DeVilbiss Healthcare France liability. In particular, using slings that are not marketed by drive DeVilbiss Healthcare France for this patient lift will not entail our liability.**

● The regular checks and inspections mentioned in the previous paragraph should be performed by a competent staff and recorded in the maintenance book.

**Out of the warranty period, our liability will not be entailed if you do not respect these controls and inspections.**

**Only spare parts and components provided by drive DeVilbiss Healthcare France can be used to repair the patient lift.**

## IX - BEFORE CONTACTING DRIVE DEVILBISS HEALTHCARE FRANCE CUSTOMER SERVICE

PROBLEM	POSSIBLE CAUSE	REMEDY
Nothing happens when one of the remote control keys is pressed.	<ul style="list-style-type: none"> <li>▶ The remote control cord is not correctly plugged in its slot.</li> <li>▶ The remote control cable is cut.</li> <li>▶ The battery pack is not supplied</li> <li>▶ The battery pack is discharged.</li> <li>▶ The cylinder connection plug is not correctly plugged.</li> <li>▶ The cylinder power cord is cut</li> </ul>	<ul style="list-style-type: none"> <li>▶ Correctly insert the plug.</li> <li>▶ Replace the remote control.</li> <li>▶ Release the red push-button</li> <li>▶ Charge the battery pack.</li> <li>▶ Correctly insert the plug.</li> <li>▶ Replace the cylinder.</li> </ul>
A cylinder produces abnormal noise.	<ul style="list-style-type: none"> <li>▶ The duty cycle 2 min./18 min. was not respected.</li> <li>▶ The pinion or the nut supporting the lifting tube is damaged.</li> </ul>	<ul style="list-style-type: none"> <li>▶ Send the cylinder for an inspection to Drive DeVilbiss Healthcare France Customer Service.</li> <li>▶ Send the cylinder for an inspection to Drive DeVilbiss Healthcare France Customer Service</li> </ul>
The cylinder works, but very slowly.	<ul style="list-style-type: none"> <li>▶ The power supply cord is not correctly connected.</li> <li>▶ There is no voltage on the mains outlet.</li> </ul>	<ul style="list-style-type: none"> <li>▶ Check the mains outlet connection.</li> <li>▶ Check the wall socket is a mains socket.</li> </ul>
The batteries do not charge.	<ul style="list-style-type: none"> <li>▶ The power supply cord is not correctly connected.</li> <li>▶ There is not voltage no the mains outlet.</li> </ul>	<ul style="list-style-type: none"> <li>▶ Check the mains outlet connection.</li> <li>▶ Check the wall socket is a mains socket.</li> </ul>
The red indicator under the small key is on..	<ul style="list-style-type: none"> <li>▶ The device requires special maintenance.</li> </ul>	<ul style="list-style-type: none"> <li>▶ Contact drive DeVilbiss Healthcare France Customer Service.</li> </ul>
The yellow control box indicator lights up.	<ul style="list-style-type: none"> <li>▶ The battery pack is half charged.</li> </ul>	<ul style="list-style-type: none"> <li>▶ Plan to charge the battery pack.</li> </ul>
No control box indicator lights up.	<ul style="list-style-type: none"> <li>▶ The battery pack is out of order.</li> </ul>	<ul style="list-style-type: none"> <li>▶ Change the battery pack.</li> </ul>
The red control box indicator lights up.	<ul style="list-style-type: none"> <li>▶ The battery pack is discharged.</li> </ul>	<ul style="list-style-type: none"> <li>▶ Immediately charge the battery pack.</li> </ul>
The red emergency lowering ring turns on the cylinder rod.	<ul style="list-style-type: none"> <li>▶ The cylinder rod is disengaged.</li> </ul>	<ul style="list-style-type: none"> <li>▶ Push the ring upwards.</li> </ul>

## X - SERVICE BOOK

**DEVICE SERIAL NUMBER :**.....

**COMPANY :**.....

This document should be photocopied and delivered to the technician every time the patient lift is maintained.  
In case of incident, the liability can only be determined if all the sheets of this book are completed

DATE	Intervention nature: Check, inspection, quotation, repairing, etc.	Intervention description	Technician's name and position

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